

MAMMOMAT Revelation

**See the difference.
Make the difference.**

siemens-healthineers.us/mammomat-revelation




SIEMENS
Healthineers 

The breast care market has become highly dynamic.

In breast screening, 2D mammography is the gold standard—but for how much longer? In recent years, many technologies have proven their potential: worldwide, digital breast tomosynthesis and other technologies are advancing. In addition, women are more informed and sensitive about dose.

Committed to supporting clinicians and women around the globe in their fight against breast cancer, we keep driving new technologies that make sense—to support fast, reliable decisions and to make examinations comfortable for women.

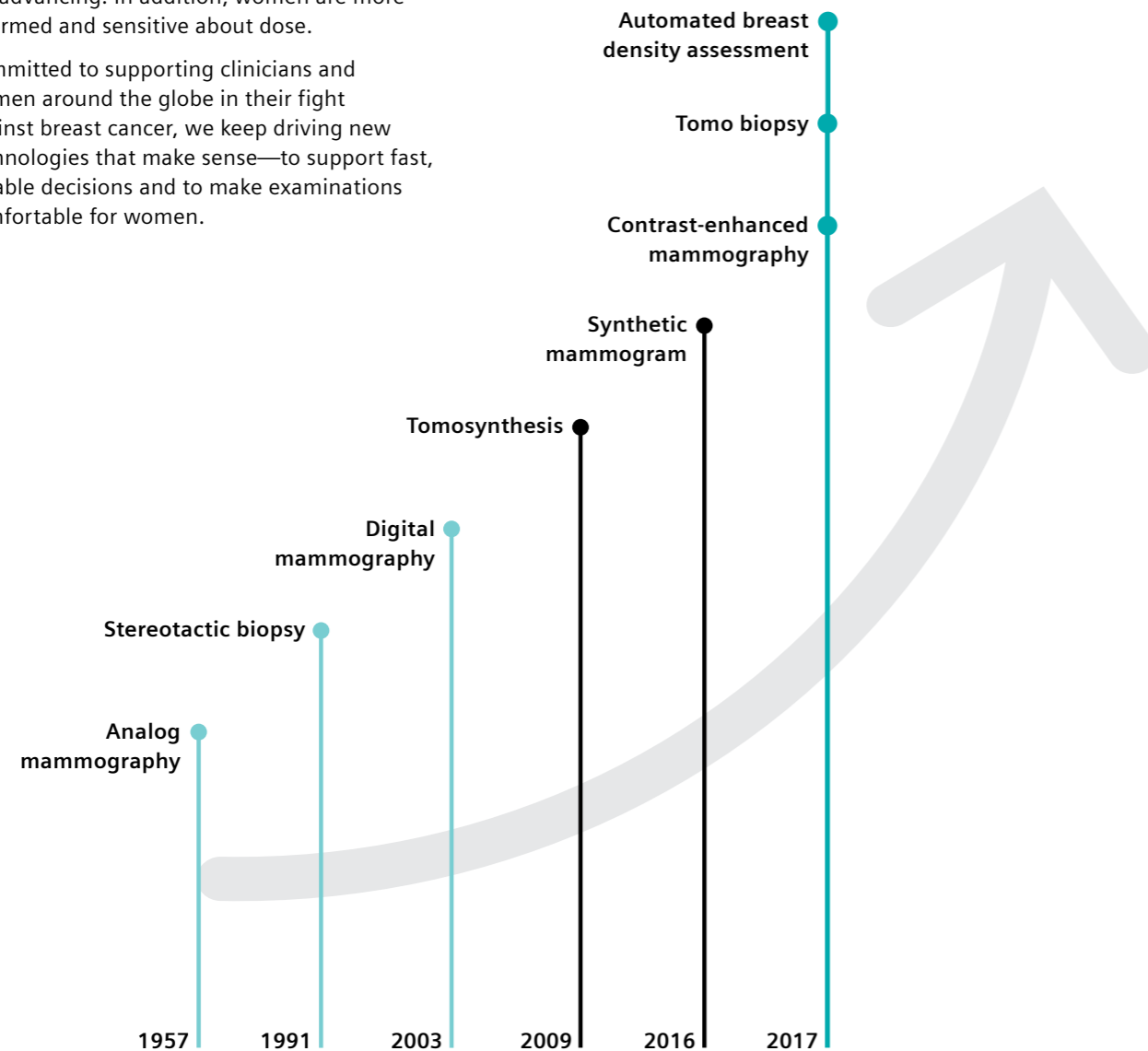

Make the difference—today and tomorrow

There are moments in a woman’s life that have the power to change its future course. In mammography, seeing the difference can make the difference to this future. Your patients put their trust in you—reveal all that matters with the new MAMMOMAT Revelation.

Reach the highest depth resolution on the market¹ with the unique 50° HD Breast Tomosynthesis—for screening, diagnostic imaging, and 3D HD Breast Biopsy. And expand your diagnostic insights with titanium-filter contrast-enhanced imaging.

Leverage precision patient care with Insight BD direct breast density measurement allowing instant risk stratification—before the patient leaves.

Discomfort or pain are key reasons why many women avoid potentially life-saving mammography exams. MAMMOMAT Revelation is designed to reduce discomfort with Personalized Soft Compression including SoftComp Paddles. And with PRIME you can perform your 2D exam with up to 30% less dose.²

Siemens Healthineers is your strong partner and the undisputed leader in imaging with 60 years of experience in breast care. Based on this expertise, MAMMOMAT Revelation is built to enhance diagnostic quality, process efficiency, and your patient’s experience.

See the difference with MAMMOMAT Revelation—and make a difference in your patients’ lives.



¹Maldera et al. (2016): Digital breast tomosynthesis: Dose and image quality assessment. *Physica Medica*, pp. 1-12.; Marshall N.W. and Bosmans H. (2012): Measurements of system sharpness for two digital breast tomosynthesis systems. *Phys. Med. Biol.* 57, pp. 7629-7650.; Rodríguez-Ruiz A. et al. (2016): Evaluation of the technical performance of three different commercial digital breast tomosynthesis systems in the clinical environment. *Physica Medica* 32, pp. 767-777.

²Compared to grid-based acquisition with Mammomat Inspiration, depending on breast thickness – L.B. Larsen, A. Fieselmann, H. Pfaff, T. Mertelmeier: Performance of grid-less digital mammography acquisition technique for breast screening: analysis of 22,117 examinations Presentation B-1025 ECR 2015



MAMMOMAT Revelation at a glance

Improve diagnostic accuracy

Overcome diagnostic challenges with the highest depth resolution available based on our 50° wide-angle technology. The outstanding image quality is the foundation for fast and reliable diagnostic decisions.

Enhance daily performance

Accelerate your examinations by reducing unnecessary steps, giving you more time—with a convenient one-click philosophy. So your staff can fully concentrate on what really matters: your patients.

Make a safe investment

In breast imaging, legal guidelines can change quickly. Get all the technological developments you need to meet the challenges of today and tomorrow—including digital breast tomosynthesis, tomo biopsy, and contrast-enhanced mammography.

Elevate patient experience

Many women are worried that getting a mammogram will be painful. That's why it's our primary objective to make mammography screenings more comfortable. For example, by slowing down the compression speed and offering a patient-friendly design.

Experience synergies in reading and reporting

Benefit from fast reading in screening and diagnostics to create accurate, high-quality reports. Our *syngo.Breast Care*¹ supports you in every step of your workflow and seamlessly adapts to existing and growing IT landscapes.

Optimize system availability

We are committed to being your trusted partner over the entire lifecycle of your system and beyond. Our service packages are designed to keep uptime high at predictable costs, lower risk, and greater efficiency.

¹*syngo.Breast Care is not commercially available in all countries. Due to regulatory reasons, its future availability cannot be guaranteed. Please contact your local Siemens Healthineers organization for further details.*

Advance screening and diagnostic results—with high diagnostic accuracy

As a radiologist, you're under pressure to reliably identify what you see in just seconds—while avoiding missing a lesion, over diagnosing or making unnecessary recalls. To support you in efficiently making confident decisions, MAMMOMAT Revelation offers the highest depth resolution¹ available combined with the latest technological trends.

Make the invisible visible

Overcome diagnostic challenges like overlapping tissue with the highest depth resolution on the market. With our 50° wide-angle technology, MAMMOMAT Revelation makes anatomical details clearly visible—with up to 3.5 times higher depth resolution compared with a narrow-angled system¹—in HD Breast Tomosynthesis² and HD Breast Biopsy.²

Standardize image quality

Variations in image quality can make reading more difficult and time-consuming. MAMMOMAT Revelation helps you achieve standardized, high image quality for each individual patient. It offers intelligent algorithms for automated and personalized parameter settings as well as unique system-guided compression.

Get more than just another view

Get unique synthetic visualization with Insight 2D and 3D³. Because it looks similar to a mammogram, Insight 2D is ideal for easy comparison with prior images.³ However, since it is created from the same dataset as the tomosynthesis slices, you reduce dose by 40%.⁴ The same data can also be uniquely displayed in 3D to speed up tomosynthesis reading. Used as an adjunct to tomosynthesis, Insight 3D supports easier analysis of microcalcifications at a glance.⁴

Rely on consistent density assessment

Visual classification of breast density according to BI-RADS standards can be challenging. In fact, more than 70% of radiologists⁵ state that they are not always confident about their categorization in each and every exam. Insight Breast Density (Insight BD) offers you 100% objective classification⁶ for instant risk stratification right at the acquisition workstation.

¹Maldera et al. (2016): Digital breast tomosynthesis: Dose and image quality assessment. *Physica Medica*, pp. 1-12.; Marshall N.W. and Bosmans H. (2012): Measurements of system sharpness for two digital breast tomosynthesis systems. *Phys. Med. Biol.* 57, pp. 7629-7650.; Rodríguez-Ruiz A. et al. (2016): Evaluation of the technical performance of three different commercial digital breast tomosynthesis systems in the clinical environment. *Physica Medica* 32, pp. 767-777.

²Option

³Insight 2D and 3D are navigational tools only and not for diagnostic use.

⁴Uchiyama N. et al. (2016): Diagnostic Usefulness of Synthetic MMG (SMMG) with DBT (Digital Breast Tomosynthesis) for Clinical Setting in Breast Cancer Screening. *Springer International Publishing Switzerland. IWDM 2016, LNCS 9699*, pp. 59-67.

⁵Data on file

⁶Dr. Andreas Fieselmann, Performance Evaluation of Siemens Volumetric Breast Density Analysis Application (internal test protocol)




The mammogram with the tomosynthesis taken on the narrow-angled system showed no obvious mass to correlate with the clinical findings. This was somewhat surprising. We expected to see some abnormality. So we repeated the tomosynthesis study on the wide-angled system. Now we could clearly see a spiculated mass posteriorly in the upper inner quarter of the left breast.

Wayne Lemish, MD, Melbourne, Australia

Clinical case

Large palpable mass posterior in the upper quarter of the left breast in a 55-year-old woman. A biopsy later confirmed the mass to be invasive ductal carcinoma grade 2.

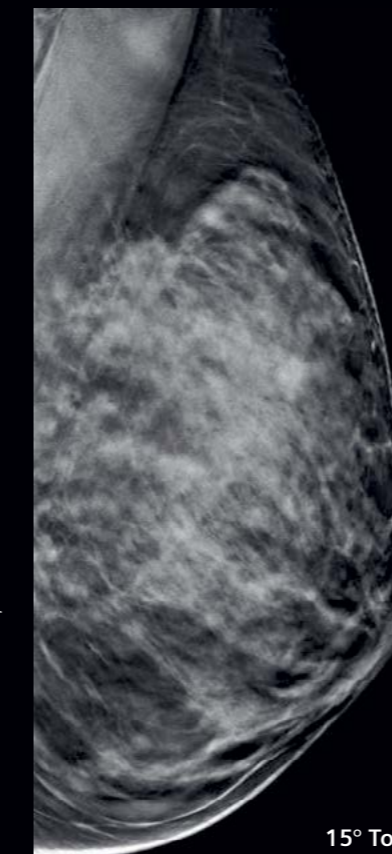

15° Tomosynthesis



50° Tomosynthesis

*Results from case studies are not predictive of results in other cases.

Elevate patient experience— with smart automation

Extremely high throughput and nervous patients are a big challenge, especially in screening. With relaxed patients, it's much easier to achieve high image quality with no motion artifacts. MAMMOMAT Revelation offers smart automation and a relaxing atmosphere—and allows you to fully focus on your patients.



Personalize compression for optimal results

Compression of the breast is typically user dependent, which can result in poor image quality and unnecessary patient discomfort. With our unique OpComp function, the optimal compression for each patient is adjusted automatically: When the paddle reaches the breast, speed is reduced. This gives staff time for proper positioning. As soon as the optimal compression for the best image quality is reached, the paddle stops—increasing patient comfort.

Personalized Soft Compression



Enhance patient comfort

A proper paddle design is crucial for correct positioning and a positive patient experience. Featuring soft edges and a breast-optimized shape, SoftComp Paddles provide better access for easier positioning and support you in achieving both excellent images and high comfort.



Reduce unnecessary steps

MAMMOMAT Revelation allows you to focus on your patients instead of the system—thanks to our one-click philosophy. With just one click, you will select the patient and start the examination. All parameters are automatically set, and the procedure moves from step to step—with no additional clicks. Easily move the system with the Single-Touch button to the next predefined examination position.

Create a relaxing atmosphere

Mammography is often associated with pain and negative experiences, resulting in nervous patients. Our Personalized Soft Compression solution combined with the calming MoodLight¹ support you in reducing discomfort and creating a better patient experience.



The rounded edges of the SoftComp Paddle significantly reduce the pressure on the area below the clavicle, which makes MLO imaging in particular much more comfortable for the patient. I also have better access to the breast, which is really helpful when it comes to positioning the patient. Especially for breasts with a very small volume, I can withdraw my hand more easily.

Astrid Reichelt, MTA
MVZ Prof. Dr. Uhlenbrock & Partners
Dortmund, Germany

Innovate care delivery—designed for excellence today and tomorrow

Patients expect individual, state-of-the-art treatment. With smart integrated features, MAMMOMAT Revelation supports you in offering personalized care. When you have the technologies of today and tomorrow, you remain at the forefront of clinical developments. And due to its modular design, the system can be adapted any time to meet your clinical needs—making it a safe investment.

Offer your patients certainty

Delivering excellent imaging for diagnostic accuracy, MAMMOMAT Revelation helps you provide greater peace of mind to both patients and referrers. In fact, 50° wide-angle tomosynthesis has been shown to provide an increased detection rate of invasive cancer of 41.5% in one of the largest studies worldwide conducted with more than 15,000 patients in screening—and with just a 1-view tomo scan.¹

Attract more women for screening

Many women are afraid of mammography because of possible pain during compression and dose application. Drive away their concerns with our unique Personalized Soft Compression. Reassure patients that they're receiving optimal care by providing low-dose examinations with up to 30% less dose² in 2D, thanks to our unique PRIME Technology.

Streamline your clinical workflow

Biopsies are not just time-consuming, they can also be painful for the patient. Streamline your biopsy workflow with the one-click targeting of our HD Breast Biopsy. Using our integrated specimen scanner InSpect³, you can decompress the patient sooner: It delivers the specimen scan within 20 seconds, without the technologist having to leave the room. This eliminates the need for an additional, dedicated specimen scanner, making MAMMOMAT Revelation a safe investment.

Stay at the forefront of clinical developments

Some difficult cases require additional diagnostic information for confident decision-making. One possible adjunct examination is MRI. However, it is time-consuming, cost-intensive, and not always available. MAMMOMAT Revelation offers the option to integrate Titanium Contrast Enhanced Mammography (TiCEM)³ as a cost-effective diagnostic alternative—reducing scheduling conflicts and the workload on other modalities.

¹Lång K. et al. (2016): Performance of one-view breast tomosynthesis as a stand-alone breast cancer screening modality: results from the Malmö Breast Tomosynthesis Screening Trial, a population-based study. Eur Radiol 26, pp. 184-190. 1-view tomo is not FDA-approved in the U.S.


²Compared to grid-based acquisition with Mammomat Inspiration, depending on breast thickness – L.B. Larsen, A. Fieselmann, H. Pfaff, T. Mertelmeier: Performance of grid-less digital mammography acquisition technique for breast screening: analysis of 22,117 examinations Presentation B-1025 ECR 2015

³Option

Expand your diagnostic insights

To make fast and confident diagnostic decisions, you need high image quality. Whether Titanium Contrast Enhanced Mammography (TiCEM), HD Breast Tomosynthesis: With MAMMOMAT Revelation you can see more diagnostic information that can be relevant for a reliable diagnosis your patients can trust.


TiCEM—benign: big round indistinct lesion, suspicious for abscess

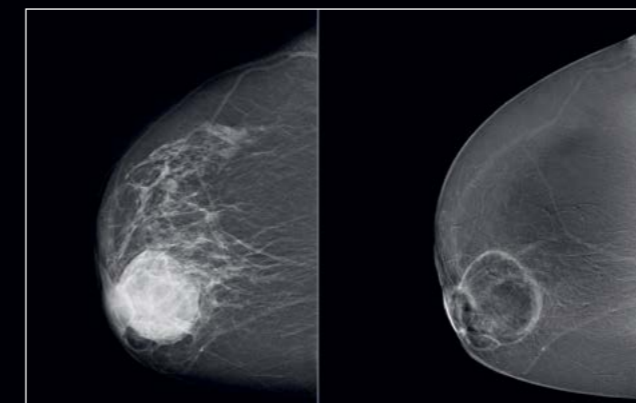

FFDM

Insight CEM


TiCEM—malignant: BIRADS 5b, ductal-invasive carcinoma

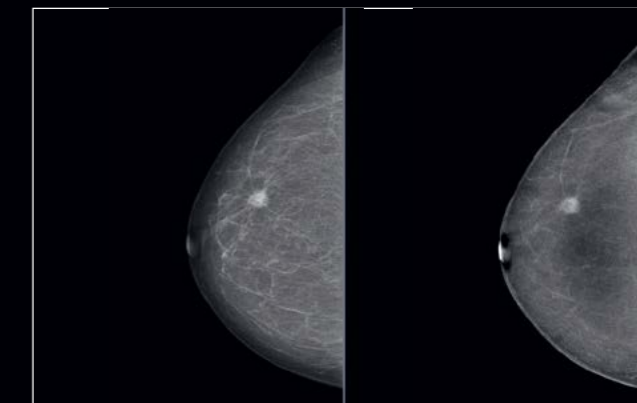

FFDM

Insight CEM

Tomosynthesis: BIRADS 5, ACR2, radial lesion, highly suspicious for malignancy

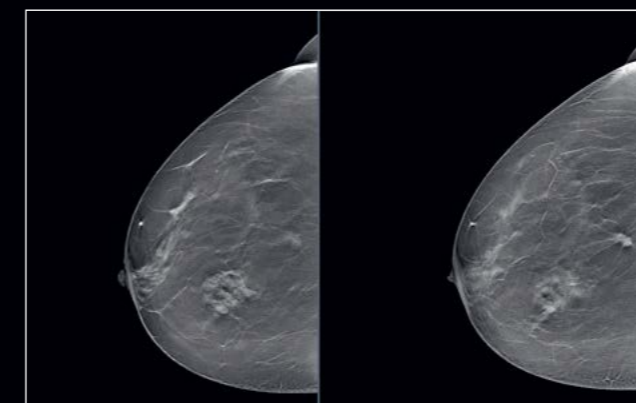


Tomo slice 24

Tomo slice 15

Synthetic view

See the difference and get the real 3D feeling with Insight 3D. Visit the MAMMOMAT Revelation website: siemens.com/mammomat-revelation

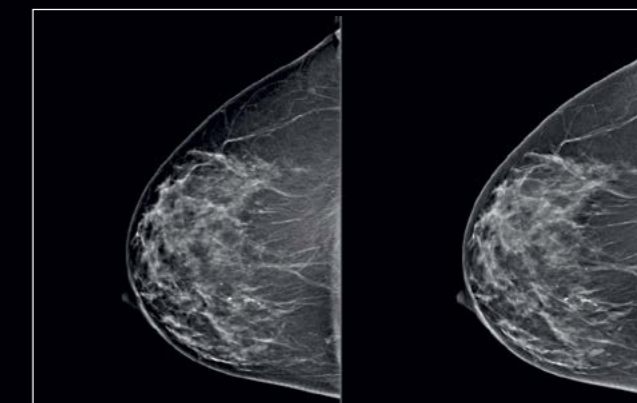

Insight 3D (rotating)

Insight 2D

Courtesy of Dr. Jise Vejborg, Prof. Dr. Detlev Uhlirbrock, and Prof. Dr. Luis Pina

Technical specifications

HD Breast Tomosynthesis

Make confident diagnostic decisions using our unique 50° wide-angle tomosynthesis. It provides the highest depth resolution on the market¹—which is key for tissue separation in 3D mammography.

Insight 2D* and 3D

Reduce dose by up to 40% when using Insight 2D instead of FFDM as an adjunct to tomosynthesis.² Because it looks similar to a mammogram, it is ideal for easy comparison with prior images. Our unique Insight 3D provides you with the real 3D feeling: See microcalcifications at a glance and easily communicate findings to peers and patients.²

HD Breast Biopsy

Profit from our unique 50° wide-angle technology that results in a target accuracy of +/- 1 mm and reduce your biopsy procedure time thanks to easy one-click targeting. And stay flexible—because you can use the biopsy device you prefer.

InSpect—Integrated Specimen Tool

Save time and costs: get your specimen scan within just 20 seconds right on the mammography system.

Insight BD—Breast Density assessment

Benefit from instant risk stratification right at the acquisition workstation with 100% objective, volumetric breast density assessment—for FFDM and tomosynthesis.

TiCEM—Titanium Contrast Enhanced Mammography

Receive additional diagnostic information in order to confidently detect or rule out lesions. TiCEM can be used as a cost-effective diagnostic alternative to MRI, while it reduces scheduling conflicts and workload on other modalities. The optimized titanium filter reduces X-ray tube load for seamless examinations.

Personalized Soft Compression

Reduce patient discomfort and achieve consistent image quality with Personalized Soft Compression: The optimal compression is automatically adjusted for each woman. The soft edges and breast-optimized shape of the SoftComp Paddles provide better access for easier positioning and higher patient comfort.

PRIME Technology

Reduce dose by up to 30%³ in 2D without compromising image quality using the world's first software-based anti-scatter solution for mammography. Sliding back the mechanical grid means there is no longer a fixed object absorbing radiation between breast and detector, so you can utilize 100% of the primary radiation—and achieve lower overall dose levels.

System specifications

Detector technology	Amorphous selenium (aSe)
Detector size	24 cm x 30 cm (9.5-in. x 12-in.)
X-ray tube anode material	Tungsten
Filter	Anode filter combinations: W/Rh, W/Ti
Swivel range	+180° to -180°, motorized, isocentric rotation with preselectable rotation angle
Swivel range of tomosynthesis mode	-25° to +25° with 25 projections
Weight of biopsy unit	< 11 lbs
X-ray release options	Control desk, hand switch, and footswitch options

*Insight 2D is a navigational tool only.

¹Maldera et al. (2016): Digital breast tomosynthesis: Dose and image quality assessment. *Physica Medica*, pp. 1-12.; Marshall N.W. and Bosmans H. (2012): Measurements of system sharpness for two digital breast tomosynthesis systems.

Phys. Med. Biol. 57, pp. 7629-7650.; Rodríguez-Ruiz A. et al. (2016): Evaluation of the technical performance of three different commercial digital breast tomosynthesis systems in the clinical environment. *Physica Medica* 32, pp. 767-777.

²Uchiyama N. et al. (2016): Diagnostic Usefulness of Synthetic MMG (SMMG) with DBT (Digital Breast tomosynthesis) for Clinical – Setting in Breast Cancer Screening. Springer International Publishing Switzerland. IWDW 2016, LNCS 9699, pp. 59–67.

³Compared to grid-based acquisition with Mammomat Inspiration, depending on breast thickness – L.B. Larsen, A. Fieselmann, H. Pfaff, T. Mertelmeier: Performance of grid-less digital mammography acquisition technique for breast screening: analysis of 22,117 examinations Presentation B-1025 ECR 2015

Discover how you can see the difference and make the difference with MAMMOMAT Revelation.



Additional products and services



LifeNet

LifeNet2 is our web-based portal that lets you manage the performance and maintenance of all your Siemens Healthineers systems 24/7. Whether you want to plan maintenance activities, monitor the status of your systems and service requests, or document service history, LifeNet gives you full control and confidence at all times. Ease your workday and stay calm, no matter what's happening around you.

syngo.Breast Care

Combine MAMMOMAT Revelation with our intelligent imaging software for multimodality reading. When you are managing high data volumes from different sources every day, process efficiency and intuitive usage is key. Boost your reading performance with advanced tools and impressively intuitive utilization. So you can view, interpret, and share images anywhere throughout your clinical enterprise for fast and comprehensive results.¹ The highly flexible modules adapt to any existing IT landscape and grow with your clinical needs, by simply adding powerful new applications.

Service and maintenance

Protect your investment over its entire lifecycle and enjoy optimized system availability, consistent image quality, and transparent total cost of ownership. For MAMMOMAT Revelation, we offer you a dedicated service package including:

- Preventive maintenance to increase system uptime
- Remote diagnosis and repair to help keep service costs down
- Corrective maintenance and comprehensive parts coverage, including tube and detector



PEPconnect—your smarter connection to knowledge

Staffing shortages, cost constraints, greater workloads—and all the while, you have to deliver an ever higher quality of care. One key to overcome these challenges is further education. With our online e-learning experience PEPconnect², you can boost your learning—anytime, anywhere, and from any device.

¹Prerequisites include: Internet connection to clinical network, DICOM compliance, meeting of minimum hardware requirements, and adherence to local data security regulations.
²Subject to country-specific availability

Why Siemens Healthineers?

Uncompromising passion for giving breast cancer the fight of its life.

In an industry where early detection changes outcomes, Siemens Healthineers is changing the game. From screening and diagnosis to treatment and follow-up, we're the company that meets every woman wherever she is on her journey to good breast health.

No two women are alike, and each care pathway is different. No matter where she is on her road to health, we're with her—and you—creating and delivering the advanced technologies that support you both at every step.

We're with her at Screening, with accurate and reliable solutions that deliver greater certainty. We're with her at Diagnosis, with innovations that offer the right answers in a time of uncertainty. We're with her at Treatment, with therapeutic options built to deliver hope. We're with her at Follow Up, to monitor every facet of her treatment and support her best possible health.

It's our belief that partnerships are built on people. With Siemens Healthineers, there is no team more passionate in the fight against breast cancer, and no group of people more committed to working with its partners to end this battle once and for all.

At Siemens Healthineers, our purpose is to enable healthcare providers to increase value by empowering them on their journey toward expanding precision medicine, transforming care delivery, and improving patient experience, all enabled by digitalizing healthcare.

An estimated 5 million patients globally benefit every day from our innovative technologies and services in the areas of diagnostic and therapeutic imaging, laboratory diagnostics, and molecular medicine, as well as digital health and enterprise services.

We are a leading medical technology company with over 170 years of experience and 18,000 patents globally. With more than 48,000 dedicated colleagues in 75 countries, we will continue to innovate and shape the future of healthcare.

The outcomes and statements provided by customers of Siemens Healthineers are unique to each customer's setting. Since there is no "typical" hospital and many variables exist (e.g., hospital size, case mix, and level of service/technology adoption), there can be no guarantee that others will achieve the same results.

On account of certain regional limitations of sales rights and service availability, we cannot guarantee that all products included in this brochure are available through the Siemens Healthineers sales organization worldwide. Availability and packaging may vary by country and is subject to change without prior notice. Some/All of the features and products described herein may not be available in the United States.

The information in this document contains general technical descriptions of specifications and options as well as standard and optional features, which do not always have to be present in individual cases.

Siemens Healthineers reserves the right to modify the design, packaging, specifications, and options described herein without prior notice. For the most current information, please contact your local sales representative from Siemens Healthineers.

Note: Any technical data contained in this document may vary within defined tolerances. Original images always lose a certain amount of detail when reproduced.

Siemens Healthineers Headquarters

Siemens Healthcare GmbH
Henkestr. 127
91052 Erlangen, Germany
Phone: +49 9131 84-0
siemens-healthineers.com

USA

Siemens Medical Solutions USA, Inc.
Healthcare
40 Liberty Boulevard
Malvern, PA 19355-9998, USA
Phone: +1-888-826-9702
siemens-healthineers.us